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Rev.03

MAINIT SILVERNITRATE MAINIT SILVERNITRATE Caustic Sticks

Medical device

PLEASE READ THE INSTRUCTIONS CAREFULLY READ THE INSTRUCTIONS FOR USE.

Always use this medical device exactly as described in the instructions for use or as directed by your physician or pharmacist. Consult your pharmacist if you need further information or advice or notice any side effects. If there is no improvement or even deterioration after repeated use, contact your physician.

CONTENTS OF INSTRUCTIONS FOR USE:

1. What are MAINIT SILVERNITRATE Caustic Sticks; what are they used for?
2. What should you pay attention to before using MAINIT SILVERNITRATE Caustic sticks?
3. How should MAINIT SILVERNITRATE Caustic sticks be used?
4. What side effect may occur?
5. How should MAINIT SILVERNITRATE Caustic sticks be stored?
6. Package contents and additional information

1. WHAT ARE MAINIT SILVERNITRATE Caustic sticks AND WHAT ARE THEY USED FOR?

Caustic sticks contain silvernitrate and potassium nitrate.

The high concentration of silver nitrate has a crusting and caustic effect. Potassiumnitrate is used as a hardening agent so that the reaction of the silver nitrate is milder. The mixture is applied to a stick just like a match and ready for use after moistening.

This medical device allows a spot application so that its action is limited to precisely defined areas.

The MAINIT SILVERNITRATE caustic sticks, come in 2 forms of application

1. As a single stick with a cap and tip of silver nitrate. This is intended for 1 treatment period of 1 person. Use by others is strongly discouraged due to contamination.
2. As a longer **single-use** stick. These are packaged in a bag of 10 or of 100.

AREAS OF APPLICATION:

The caustic sticks are used in the treatment of warts, removal of excess tissue (created in the healing process of wounds).

2. WHERE TO PAY ATTENTION TO BEFORE USING MAINIT Caustic sticks WITH SILVER NITRATE?

Caustic sticks must not be used if you are allergic to silver nitrate or potassium nitrate. It may take up to 24 hours for the treated area to become discolored.

WARNINGS AND PRECAUTIONS:

- Any unusually discolored or unusually shaped skin changes that are not unambiguously identifiable as 'wild flesh' or as common warts, the MAINIT SILVERNITRATE Caustic Sticks should only be used under the supervision of a physician.
- Don't touch the tip (etchant) of the MAINIT SILVERNITRATE Caustic Sticks with your hands.
- Protect your clothing and other objects while using of the MAINIT SILVERNITRATE Caustic Sticks. Silver nitrate causes brown-black stains that cannot be removed with common cleaning or laundry detergents.
- Healthy skin that inadvertently comes into contact with the tip of the MAINIT SILVERNITRATE Caustic Stick should be carefully cleaned with running water and a sponge. Nevertheless, a harmless dark discoloration of these skin areas may occur. The dark discoloration will fade again after about 10-14 days.
- If the MAINIT SILVERNITRATE Caustic stick has come into contact with the eye immediately rinse under running water with the eyelid open for at least 5 minutes and then consult a physician.
- Do not use the caustic stick on sensitive areas of the body (such as eyes, genitals).
- When used around the mouth, be careful not to swallow any parts of the tip.
- Any serious event related to the medical device should be reported to the manufacturer, and the competent authorities of the Member State where the user is located.

CHILDREN: Use in children under 3 years of age should only be on the advice of a physician.

Pregnancy AND Lactation: If you are pregnant, nursing, suspect that you are pregnant or plan to become pregnant, you should consult your physician or pharmacist before using this medical device.

DRIVING SAFETY: The use of MAINIT SILVERNITRATE Caustic Stick does not affect the ability to drive.

3. HOW SHOULD MAINIT SILVER NITRATE Caustic sticks BE USED?

Always use this medical device exactly as described in the instructions for use. When in doubt, consult your physician or pharmacist.

HOW TO USE:


- For external use;
- Clean area to be treated;

See drawing below;

- Fig1. Put a drop of clean water on a non-absorbent, clean, firm surface, such as an upside-down glass on a table.
- Fig2. Place the tip of the Caustic stick in the drop of water, move the tip very gently (about 2 seconds) in the water.
- Fig3. The stick is ready for use.
- Fig4. **DO NOT** let the water or the tip of the stick come into contact with healthy skin, clothing, furniture or other surfaces. Protect healthy skin around area to be treated with a thin layer of petroleum jelly.
- Fig5. **Gently** dab with the tip of the Caustic stick on the area to be treated. Do **NOT** use pressure. If the tip breaks, make sure that all parts are removed with a paper towel.
- Fig6. Let the treated spot dry thoroughly, the spot will turn black within a short time, and be shed in the following days.

Repeat the treatment every 2-3 days until the desired result is achieved. Multiple treatments may be required. Do not use it for more than 14 days. If there is no improvement, consult a physician.

Use a paper towel to dry the tip of the stick and the remaining water.

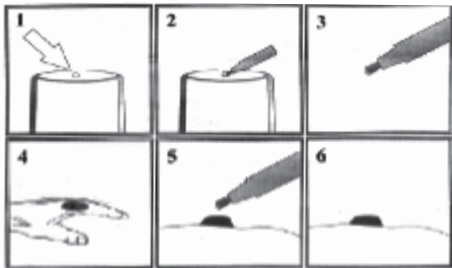IN CASE OF OVERDOSE OF MAINIT SILVERNITRATE Caustic sticks:

Excessive use (without waiting for change) may cause irritation or inflammation around the treated area. In such cases, consult your physician before further use

4. WHAT SIDE EFFECTS MAY OCCUR?

There may be hypersensitivity reactions or local irritation of the skin such as itching, red discoloration, blisters or a slight burning sensation. If you notice any side effects, consult your physician or pharmacist. This also applies to side effects not listed in the directions for use.

5. HOW SHOULD MAINIT SILVER NITRATE Caustic sticks BE STORED?

- Store this medical device in the original packaging to protect the contents from light.
- Store the packaging in a dry place. Close the package carefully after open until the next use.
- Store this medical device out of the reach of children.
- You should not use this medical device after the expiration date.
- The expiration date is indicated on the back of the package.
- After use, the sticks can be disposed by way of regular household waste.

6. PACKAGE CONTENTS AND ADDITIONAL INFORMATION:

Available in packs of:

- Once 1 piece 40% caustic stick; Red stick packed in cardboard box; Mainit artnr 226504 (one person, multiple use)
- Once 1 piece 95% caustic stick; Blue stick packed in cardboard box; Mainit artnr 226511 (one person, multiple use)
- Once 100 pieces 75% caustic stick; Longer sticks packed in a bag of 100 in a plastic tube; Mainit artnr 226505 (single use)
- Once 100 pieces 95% caustic stick; Longer sticks packed in a bag of 100 in a plastic tube; Mainit artnr 226506 (single use)
- Once 10x10 pieces 75% caustic stick; Longer sticks packaged in 10 bags of 10 in a cardboard box; Mainit artnr 2265051 (single use)
- Once 10x10 pieces 95% caustic stick; Longer sticks packaged in 10 bags of 10 in a cardboard box; Mainit artnr 2265061 (single use)

More information can be found on our homepage www.mainit-bv.nl.

MANUFACTURER:

 Mainit-BV
Roode Wildemanweg 5
1521PZ Wormerveer
The Netherlands
info@mainit-bv.nl
www.mainit-bv.nl

VERSION OF THE IFU: 20 October 2025

Rev.03